
TUBING WITH HANDLES

INSTRUCTION MANUAL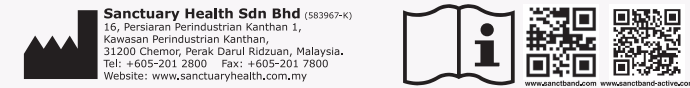

Date of issue: 2025-09-08

🇬🇧 Safety Instructions – Please Note

- Natural product with reliable linear resistance behaviour. Sanctband | Sanctband Active is a functional piece of exercise device for physical therapy, muscle increase and fitness.
- ATTENTION: Sanctband | Sanctband Active are made from natural latex which can cause allergic reactions.**
- When using Sanctband | Sanctband Active, always follow the recommendations of your doctor or therapist.
- Children may use Sanctband | Sanctband Active only under adult supervision and after a therapy-accompanying introduction. Not suitable for infants.
- Check Sanctband | Sanctband Active for damages (holes, cracks, cuts) prior to every use. Do not use damaged tubings, replace them instead.
- Do not overstretch Sanctband | Sanctband Active ($F \geq 200\%$), do not create too much tension. Choose the tubing that matches your physical condition and muscle strength.
- Be careful with the exercises at head height! Keep away from eyes, the tubing could slip out of hand or holder and snap in the face!
- Ensure the tubing cannot release from door frame or loosen from the chair or other objects. Ensure that door is shut while exercising.
- Sanctband's | Sanctband Active's Recommended Cleaning Method: Clean with wet clean cloth.
- Shelf Life: 5 years from the date of manufacture if unopened and unused.
- 11. Disposal of the devices:** Follow local disposal procedure as per user facility.

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ATTACHING A DOOR ATTACHMENT	BICEPS CURL	LATERAL RAISE	SQUAT	TRICEP KICKBACK
GLUTE BRIDGE	SEATED LOW ROW	STANDING GLUTE KICKBACKS	CRUNCHES	ROTATIONAL CRUNCHES